

BLOOD TEMP 10



Product Description

The Blood Temp 10 temperature indicator is a simple quality assurance for blood temperature, accurately monitoring the temperature of Blood Bags during storage and transit. If temperature breaches 10°C, the temperature indicator will visually alert handlers.

The alarm is irreversible, preventing tampering, and easy to use. Requiring no special handling prior to use, the Blood Temp indicator is shelf stable and can be stored at room temperature.

The Blood Temp temperature indicator is compliant to all FDA and AABB requirements: FDA CFR600. 1 5 (a) for monitoring blood temperature. FDA 510(k) #BK150289.

Industries Served

- Pharmaceutical



1 Push the Button



Firmly squeeze the activation button until you feel a pop, then release.

2 Verify Activation




Arming window will change from yellow to dark green immediately. Dark green means activated. If not, go back to step 1 and repeat.

3 Apply to Blood Bag



Remove adhesive liner and adhere activated indicator to the lower third of the blood bag.

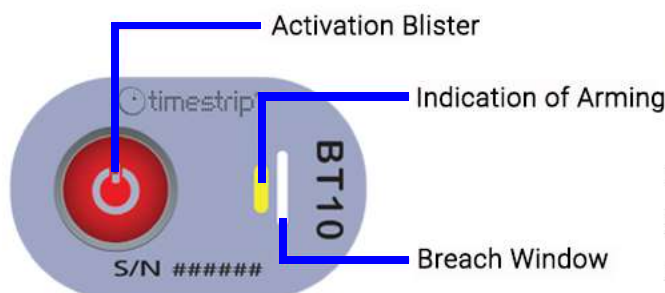

Blood Temp 10 clearly indicates when temperature breaches 10°C. Colour change is irreversible and cannot be reset or changed.



At-a-glance temperature monitoring

Upon activation, Blood Temp 10 instantly monitors blood temperature.



STREAM PEAK INTERNATIONAL PTE LTD
Tel: +65 6610 7921 | Fax: +65 6862 2520
Email: enquiry@streampeak.com.sg

Valuing Relationships, Protecting Goods
Address: 26 Tuas South Street 3, S(638026)
Website: www.streampeak.com.sg

Key Features

- Enables quick accept/reject decisions
- Irreversible evidence
- Cost-effective
- Easy to interpret
- Aids in compliance with regulatory guidelines
- Blood Temp 10°C meets all FDA and AABB requirements:
FDA CFR 600. 1 5 (a) for monitoring blood temperature. FDA 510(k) #BK150289



Product Specifications

Technical Specifications	
Monitored Temperature	10°C / 50°F
Temperature Accuracy	± 0.5°C / ± 1°F
Storage Conditions	Store in a cool, dark place. Recommended 22°C / 72°F at 20-80% RH
Shelf Life	2 year from date of manufacture
Size	1.25in x 0.75in / 32mm x 19mm

Related Products



Thermal Insulator



Ice Peak Ice Gel Pack



WarmMark



ColdMark



STREAM PEAK INTERNATIONAL PTE LTD
Tel: +65 6610 7921 | Fax: +65 6862 2520
Email: enquiry@streampeak.com.sg

Valuing Relationships, Protecting Goods
Address: 26 Tuas South Street 3, S(638026)
Website: www.streampeak.com.sg